EZ INJECTOR®

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Description

In order to use it safely, you must read the instructions before using it.
The installation contents are used correctly. For the convenience of future use, please keep the instructions properly.
If you want to give the product to others, you must hand it over with the instruction manual.

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■ 产品介绍 (Product Introduction)	----- 1	■ 屏幕构成及决定 (Screen composition and decision)	- 8
■ 注意事项 (matters need attention)	----- 2	■ 充电方法 (Charging method)	----- 9
■ 产品构成 (Product composition)	----- 3	■ 产品配置 (product configuration)	----- 10
■ 各部名称 (Names of ministries)	----- 4	■ 质量证书 (certificate of quality)	----- 11
■ 使用方法 (method of application)	----- 5		

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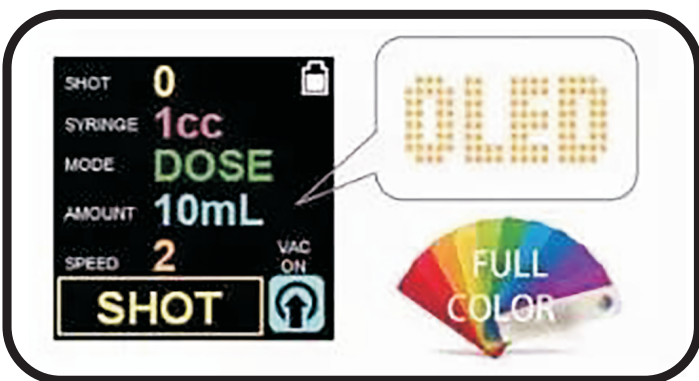
- 在可以真空吸入的手柄上安装了负压，使水光注射更强力，更方便。
 - 取下负压手柄，单独使用主机的话，还可以进行玻尿酸注射。
- **Detachable butt-joint negative pressure handle**
- A negative pressure is installed on the handle capable of vacuum suction, so that Hydrophotic injection is more powerful and convenient.
 - Take off the negative pressure handle, and if you use the main engine alone, you can also for injection of hyaluronic acid.

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- 使注入时的药物损失降低到最小化的真空吸入构造。
 - 真空的吸入压力可以在针头插入后带有自动负压针刺功能。
- **Special negative pressure multi-needle**
- Reduce the loss of injected drugs to a minimum vacuum suction structure.
 - The suction pressure of vacuum can be self-contained after the needle is inserted Dynamic negative pressure acupuncture function.

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■ For safe use, observe the following precautions after knowing the contents of the manual.
Matters needing attention include general matters and transportation Please refer to the corresponding pages for details of operation and usage.

- When using disposable products (such as needles) packaged in sterilized state, please confirm whether the outer packaging is damaged. In case of damage, please Do not use.
- Do not use or store near inflammable and explosive articles.
- Please do not disassemble or modify the product at will.
- Do not touch or operate the main unit and charger with wet hands.
- Be careful not to damage the needle when installing the syringe.
- Before and after using the product, wipe the main machine and parts with a dry cloth, and always keep the machine clean and sanitary.
- Needles are disposable products, please do not reuse them
- Please confirm the charging state before use, and use it after being fully charged.
- After moving from a cold environment to a warm environment, condensation will obviously occur due to temperature changes, so please place it indoors. Time, to adapt to the temperature no longer produce condensation phenomenon or water droplets disappear before use.
- Please be careful not to flow the injection solution into the machine. The solution flowing into the machine will damage the motor and internal accessories.
- Do not exert unreasonable force on the buttons, motors and charging interfaces of the product, which may cause machine failure.
- Drugs or foreign bodies attached to the screw connection of syringe and connector will harden, which is difficult to separate. Please do not Let the screw connection be stained with medicine or foreign matter. If it is stained with drugs or foreign bodies, please wipe it with alcohol in time.
- If the product is damaged due to falling or external force, please check the operation before use and use it without any abnormality.
- Please keep a steady grasp during injection to prevent the host from falling.
- Do not wash with water. Failure caused by flooding is not covered by warranty.
- The charger must use the special charging equipment provided. The use of a non-dedicated charger or adapter can easily cause damage to the machine.
- Please do not connect the USB data cable of computer or other electronic equipment at the charging interface behind the host. This machine can only be connected to special purpose charger

产品构成PRODUCT COMPOSITION

①主机
Host

②负压手柄
Negative pressure handle

③充电底座
Charging base

④注射器固定头 (2种)
Injector fixing heads (2 types)
5.10cc

1.3cc

⑤注射活塞 (通用)
Injection piston (universal)

1cc.3cc.5cc.10cc

⑥充电器
charger

⑦充电转换头
Charging conversion head

⑧过滤嘴
Filter tip

⑨负压管 (2个)
Pet-name ruby
negative pressure pipe (2)

⑩ 正品原装负压针头(标配一个负压5针针头)
Attending negative pressure needle
(one as standard Negative pressure 5 needles)

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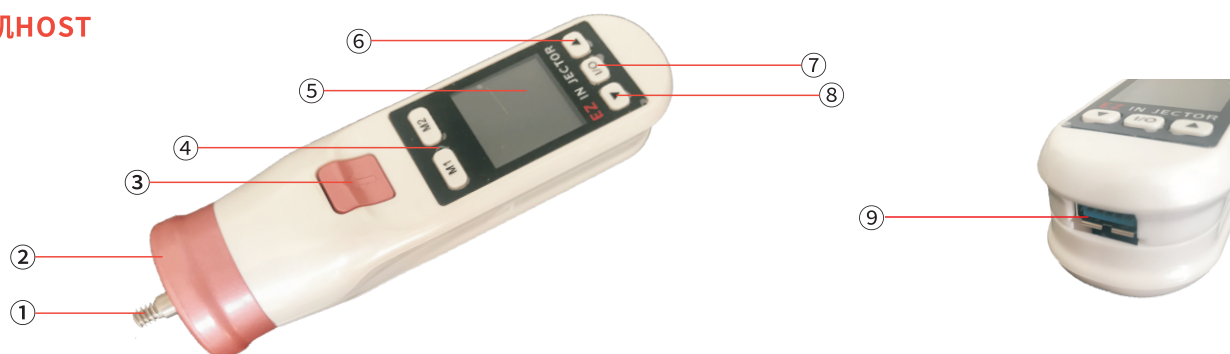
■ Component name and function

- ①Host: the main body of injection function.
- ②Negative pressure handle: it is installed on the main engine to perform negative pressure function.
- ③Charging base: the host or negative pressure handle is placed for charging.
- ④injector fixing heads (2 kinds): connect and fix 1·3cc and 5·10cc with the host computer.
- ⑤Injection piston : installed on the injector to push the injection liquid.
- ⑥Charger: use a special charger to charge the polymer lithium ion battery in the host.
- ⑦Charge converter: make the charging base directly connect with the host or the converter head of negative pressure handle.
- ⑧Filter tip: it is installed on the negative pressure handle to prevent foreign matter from inhaling when in use.
- ⑨Negative pressure pipes (2 pieces): installed between the negative pressure needle and filter tip, which plays the role of vacuum suction.
- ⑩ Negative pressure needles: 5-frequency negative pressure needles, sterilized and packaged
- ⑪One 5ml needle tube

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- ① **Injection piston:** press the plunger of the syringe to inject the contents.
- ② **connector:** connect the syringe fixing head.
- ③ **Injection key:** inject drugs.
- ④ **storage keys (M1,M2):** save or output the set value.
- ⑤ **OLED:** a screen that displays the operating status and set values.
- ⑥ **Forward key:** advance the injection piston or change the menu setting value.
- ⑦ **Power key:** switch power supply or change menu items. (Press and hold for more than 2 seconds, and the power is turned off. If there is no operation within 5 minutes, the power will be turned off automatically.)
- ⑧ **Back key:** make the injection piston back or change the set value. Press the back button again during the back operation to stop.
- ⑨ **Connecting port:** connect negative pressure handle or charging base.

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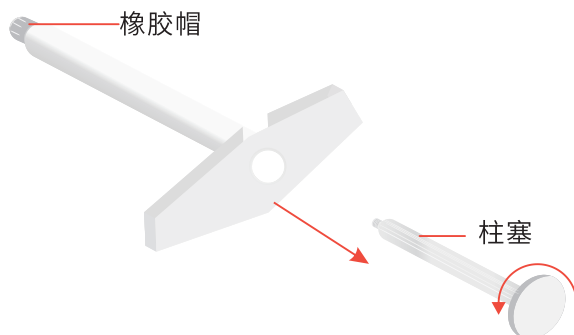
Prepare syringe

As shown in the figure, the plunger is now rotated counterclockwise

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Install the injection piston and syringe

According to the syringe to be used, the injection piston can be screwed on.

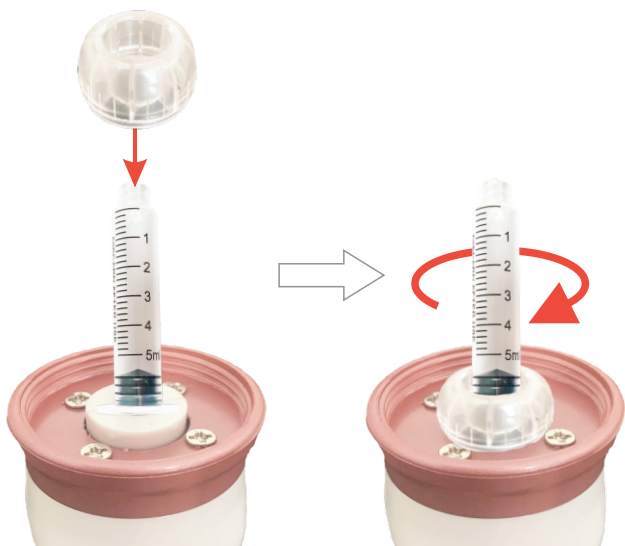
⚠ 注意 Note

- 注射活塞与注射器种类不符合的状态下连接主机，则会导致无法正常工作。
- 请在电源关闭的状态下安装注射活塞和注射器。
- If the injection piston is connected to the host in a state where the type of injector does not match, it will lead to the failure of normal operation.
- Please install the injection piston and syringe with the power off.

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将注射器固定头放在主机上按顺时针方向拧上
Install the syringe fixing head
Put the syringe fixing head on the main machine
and screw it clockwise go up

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将注射器的橡胶帽取下，安装上单针头
或者多针头后，按电源键即可开机。
Install needle and start up
Remove the rubber cap of the syringe and
install a single needle or after multiple needles,
press the power button to start the machine.

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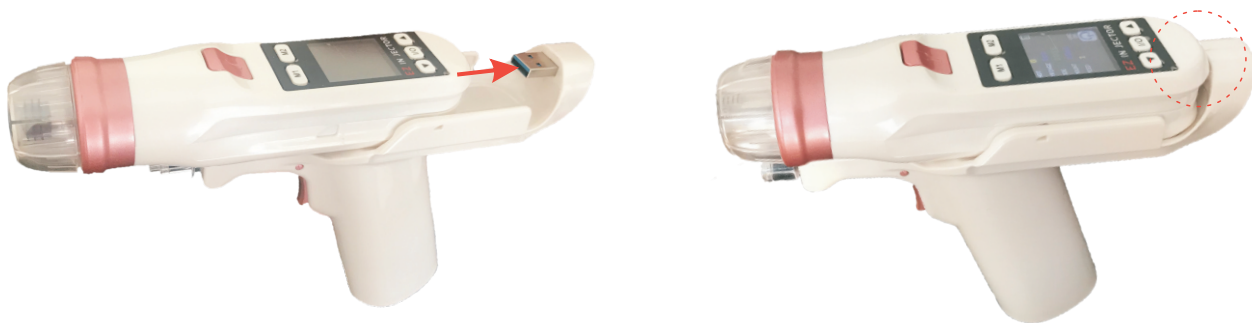
将主机的尾部从负压手柄的前段开始推入。
Install the host on the negative pressure handle
Push the tail of the main engine from the front section of the negative pressure handle.

⚠ 注意 Note

- 安装或者拆下时，要在电源关闭的状态下进行，防止按到不必要的按键而运作。
- When installing or removing, it should be carried out with the power off, so as to prevent the operation by pressing unnecessary keys.

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Install filter tip

Push the filter tip on the filter socket of the negative pressure handle.

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Installation of injector and injector fixing head

After installing the syringe, plug the fixing head and fix it on the main on board. Please refer to p.5~6 for installation method.

! 注意Note

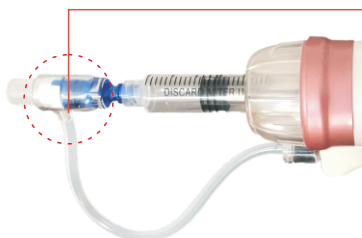
- 过滤嘴安装好了，运作时才不会漏气。
- When the filter tip is installed, it will not leak during operation.

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
Needle and hose installation

Install a negative pressure needle in the syringe, and connect the needle with the filter tip with a hose.

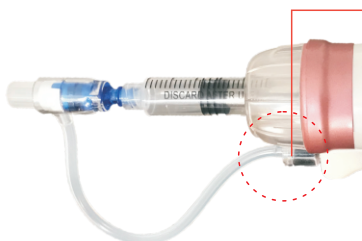负压针头的负压管接口
Negative pressure pipe connection
port of negative pressure needle

过滤嘴的负压管接口
Negative pressure pipe
connection port of filter tip

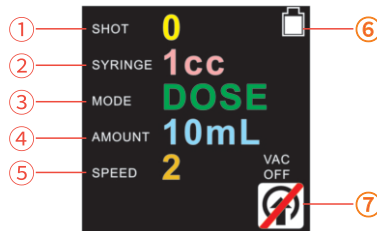
! 注意Note

- 请使用产品配有3mm(内径) x5mm(外径) 负压硅管。如用其他规格的负压管则会降低吸力。
- Please use the product with a negative pressure silicon tube of 3mm (inner diameter) and 5 mm (outer diameter). If you use negative pressure tubes of other specifications, you will Reduce suction.


屏幕构成及设定 SCREEN COMPOSITION AND SETTING

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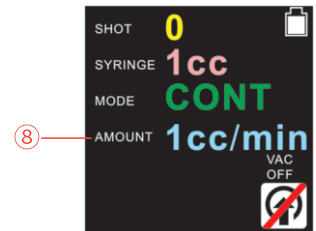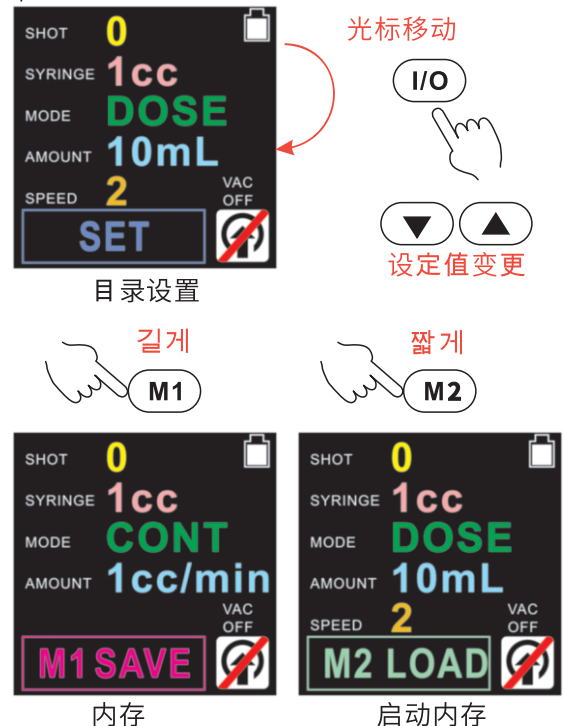
■ Menu settings

- When the power button (I/O) is pressed, it switches to the menu setting together with the SET flag Picture. Press the power button (I/O) to move the cursor to the next set value.
- Press the back key (▼) to decrease the set value, and press the forward key (▲) to increase the set value
- To exit the menu setting screen, press the power button (I/O) repeatedly. Until the SET capacity flag and cursor box disappear.

■ Setting range of different syringes

- 1cc is 10UL~500UL; 10UL per gear
- 3cc is 25UL~500UL; 25UL per gear
- 5cc is 50UL~1000UL; 50UL per gear
- 10cc is 50UL~1000UL; 50UL per gear

■ Memory key function

- Equipped with two kinds of storage keys (M1,M2), which can easily arouse two kinds of keys Set values.
- After setting the required configuration, long press the storage key for more than 2 seconds will display "M1" Save "or" M2 save ".
- When using, press the store key briefly, and "M1 load" or "M2 load" will be displayed. Convert to the stored settings.

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- 将充电器插入100~240V AC的充电底座，把插头与充电底座后面的充电插口连接。
- 主机和安装在主机上的负压手柄插在充电底座上即可充电。
- Connection between special charger and charging base
- Insert the charger into the charging base of 100~240V AC, and connect the plug with the charging socket behind the charging base.
- The main machine and the negative pressure handle installed on the main machine can be charged after being inserted into the charging base.

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- 注射器固定头使用后，请用浸泡在酒精里或者用酒精棉花擦拭。
- 其他主机部分请勿浸泡在酒精里，用干布轻轻擦拭表面即可。
- Disinfection and cleaning
- After using the syringe fixing head, please wipe it with cotton soaked in alcohol or alcohol.
- Do not soak other host parts in alcohol, and gently wipe the surface with a dry cloth.

⚠ 注意 Note

- 使用后，请在注射活塞后退完全停止后保管。如在活塞前进的状态下保管时容易折断。
- 使用过的药物或者异物粘在注射活塞时无法准确地推送药物。
- 使用前，确认是否有药物或异物，如有异物请用酒精擦拭干净。
- After use, please keep it after the retraction of the injection piston stops completely. Such as easy to break when the piston is kept in the forward state.
- When used drugs or foreign bodies stick to the injection piston, the drugs cannot be pushed accurately.
- Before and after use, confirm whether there are drugs or foreign bodies. If there are foreign bodies, please wipe them clean with alcohol.

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■ Product name: EZ INJECTOR

■ Model: EZ-1000

■ Main functions: Assisting the quantitative and constant-speed injection of hyaluronic acid and other products

■ Power supply: built-in lithium battery 7.4V, 850mAH

■ Charger: lithium battery 8.4V DC/500mAmax output, AC100~240V(50/60Hz) input

■ Charging: After being fully charged, use negative pressure function for about 2000 times at most

■ Power consumption: maximum 4W

■ Material: the main machine, negative pressure handle and charging base are made of plastic, and the injector fixing head and filter tip are made of polycarbonate for injection. The piston is made of nickel plated material, and the negative pressure pipe is silicon pipe.

■ Weight: main machine 220g (including injection fixing head), negative pressure handle 100g

■ Size: Host: -153x38x38mm

Negative pressure handle-170 x 70 x 48 mm

■ Display: 1.5-inch full color OLED

■ Power supply operation: switch manually according to the button or shut down automatically if there is no operation within 5 minutes

■ Mode: constant speed (continuous), quantitative (dose)

■ Injection setting: 5 degrees of constant speed mode,

■ Basic accessories:

1) negative pressure handle

2) There are two kinds of injection pistons: 1cc (basic installation), 3cc, 5cc and 10cc for general use

3) Charging base

4) Special charger

5) There are two kinds of syringe fixing heads: 1~3cc, 5~10cc

6) Charging adapter

7) Two negative pressure pipes (3x5 silicon)

8) One filter tip

9) One negative pressure 5-needle needle

10) A 5ml needle tube

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* The products shall be subject to the physical objects